The OV-Watch® sensor is a key component of the technology. It is filled with a non-allergenic gel that slowly releases through the two white porous filters (called frits). The gel mixes with the perspiration on the surface of your skin to form electrical contact between your skin and the sensor (black dot). The sensor is specially designed to detect chloride ions and measure the electrical properties of those ions. The sensor is designed to last one full cycle. **A new sensor must be used for each cycle.** There are generally only two ways the sensor can under perform; a) dried up frits, and b) protective tape not being removed.

**Dried Up Frits** - The drying problem is often difficult to detect. If everything about the watch appears normal and you are still not getting an OK reading in Test Mode, chances are the frits have become too dry to facilitate a reading. To solve this problem, simply **remove the sensor** (this will NOT affect your current cycle) and place it in a VERY shallow pool of water for about 90 seconds. A small plate or saucer works ideally. Cover just the surface of the plate with water and put the sensor (frits side down) on the plate. Be careful not to get water inside the sensor. Dry off the sensor with a soft dry cloth and place it back on the watch. Try a test mode reading after you have been wearing the watch for several minutes. If you do not get the OK screen in Test Mode, you may have a bad sensor. Contact Customer Service at 1-866-OVWATCH or 408-987-5510 for assistance.

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**Remember, you MUST be in the first 3 days of your period to start using OV-Watch®. If you are already past your third day, you should skip the current menstrual cycle and wait for the next one.**

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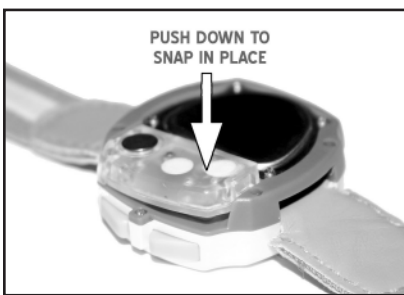
- **Remove the sensor from the foil pouch.**

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- **Peel off the red protective tape and discard. Failure to remove tape will cause OV-Watch to NOT work.**

- Pelar la cinta roja que protege y bótelo. Malogro de pelar la cinta causara que el Reloj-OV ® NO trabaje.
- Détachez la bande adhésive rouge et la jeter. (Si vous oubliez d'enlever la bande adhésive OV-Watch® ne fonctionnera pas).

- **Turn watch over so the back is facing up.**

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- **Hold sensor so that the large black circle is facing you. (Black circle towards top and 2 white circles towards the bottom).**

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- **Insert one end (black circle end or white circle end) of the sensor into the empty sensor slot on the back, left side of the watch.**

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- **Push down on the opposite end until you hear the sensor snap into place. The sensor is now attached and ready to take readings.**

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- **Place your thumbnail in-between the sensor and the metal battery cover and slide the sensor out of its slot AWAY from the battery cover.**

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- **Used sensor can be disposed of into a trash receptacle.**

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#### CUSTOMER SERVICE & TECHNICAL SUPPORT

Customer Service hours of operation are 9:00 am to 9:00 pm eastern standard time. Monday to Friday except on nationally recognized holidays. The toll free number is 1-866-OVWATCH or 408-987-5510. You may also obtain assistance via email at [customer-care@ovwatch.com](mailto:customer-care@ovwatch.com) OR [techsupport@ovwatch.com](mailto:techsupport@ovwatch.com)

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